

HOW TO CLEAN & DISINFECT RESUSABLE PROTECTIVE EYEWEAR (GOGGLES)



1. Perform hand hygiene. Put on gloves.
2. **CLEAN:** Using Accel Wipe or Cavi Wipe and a **rub and scrub motion**, clean the goggles. Work from the inside to the outside.
3. **DISINFECT:** With a second wipe, wipe all surfaces of the eye protection.
4. **DWELL TIME:** Place on clean surface (e.g. clean paper towel) and allow to air dry (3 mins for Cavi Wipe, 1 min for Accel Wipe).
5. **RINSE:** rinse any remaining residue from goggles with clean water.
6. **DRY:** Allow goggles to air dry, or dry with clean absorbent towel.